



## Instructions for Use - Lead Table Shield

SKU: LB-LTS-3636

### Cleaning:

- Clean with a damp cloth and mild detergent. If you need to scrub the shield, use a soft toothbrush
  - Avoid immersing the shield in water or putting it under a faucet.
  - Avoid harsh cleaning agents like bleach, IPA/ethanol .
  - Do not machine wash.

### Disinfecting:

- Water based QAT's – or quaternary ammonium cleaners/disinfectants – are acceptable provided they are substantially alcohol-free and made up in dilute water-based forms. Follow manufacturer's instructions for make-up/ dilution. The fabric surface should then be wiped with a dampened QAT cloth, then allowed to dry.
  - Do not autoclave.

### Storage:

- Shield should be stored on a hanger, or stored flat.
- Do not fold! Folding can lead to cracks in the protective material.