

8-Peg Spyder Apron Rack with Glove Holder, RAR-SAR8G

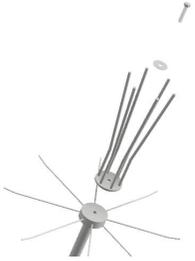
Tools required.

- Phillips Screwdriver
- 3/16" Allen Wrench
- 7/16" socket or adjustable wrench

Instructions

Step	Description	Picture
1	Begin with locating the base. Place the base upside down.	
2	Attach the Caster Wheels to the base. Attach the caster wheels by pushing the stem into the holes at the ends of the base's legs.	
3	Slide the bushing on the the end of the rod that has the stopper.	
4	Insert the Rod and the bushing into the top of the base. On the bottom of the base place a fender washer on the small bolt and tighten the bolt to the rod.	

8-Peg Spyder Apron Rack with Glove Holder, RAR-SAR8G

5	<p>Place the Glove Holder on the top of the Peg Support. Bolt both part to the rod. A fender washer should be used to place on the top of the Glove Holder.</p>	
---	---	---

