



Instructions for Use Phillips Safety Apron Racks

Models: RAR-P*, RAR-MAR*, RAR-MSA10, RAR-M10S, RAR-SAR*, RAR-F5S

Use: Hang leaded apron on hanger (or peg) depending on model.
Limit one apron per peg/hanger. Do not overload.

Cleaning:

Clean apron rack with a mild detergent or cleaner to remove any dirt or debris. Do not use abrasives. Water temperature should be at least 110 degrees.

Disinfecting:

Apron racks can be disinfected with most standard hospital grade disinfectants containing QATs, IPA or Hydrogen peroxide.

Warranty:

One year against manufacturer defects. Does not include normal wear and tear or overloading of hanging points.

Manufactured by:
Phillips Safety Products
123 Lincoln Boulevard, Middlesex, NJ 08846
732-356-1493
service@phillips-safety.com

December 2020